



# TAURI

Quick Temperature Test Tablet

## Quick Start Manual

The Tauri Temperature-Check Tablets are designed to detect temperature anomalies. It's important to understand there are many factors, including environmental and physiological that can impact a person's surface temperature reading. Skin surface temperature vs actual core body temperature may differ either way. The Tauri Temperature-Check Tablets must be operated in accordance with the manufacturer's user guide. Tauri Temperature-Check Tablets are not intended nor designed to diagnose or detect medical conditions including, but not limited to, viruses or other illnesses. The Tauri temperature-check tablets should only be used to detect variations in surface temperature. In the event that an elevated skin temperature is detected, the subject should be advised to check temperature with an approved medical thermometer and the finding confirmed. Absence of an elevated skin temperature does not preclude a fever.



# Content


Power connection -----	3
Software set up -----	4 - 10
Installation (Wall Mount) -----	11 - 12
Installation (Table Stand FIX027) -----	13
Installation (Table Stand FIX028) -----	14
Installation (Rotatable Table Stand FIX030) -----	15

## 1. Power Connection


Connect power-in to the external female power cable of TAURI.




## 2. Software Setup

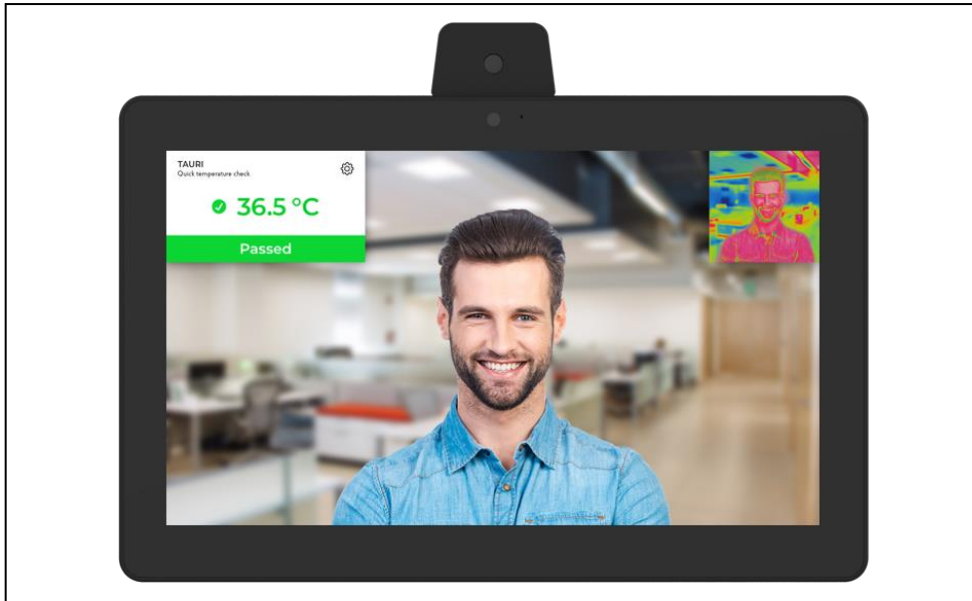


2.1 Press the “setting” icon on the top left corner and enter password “0000”.

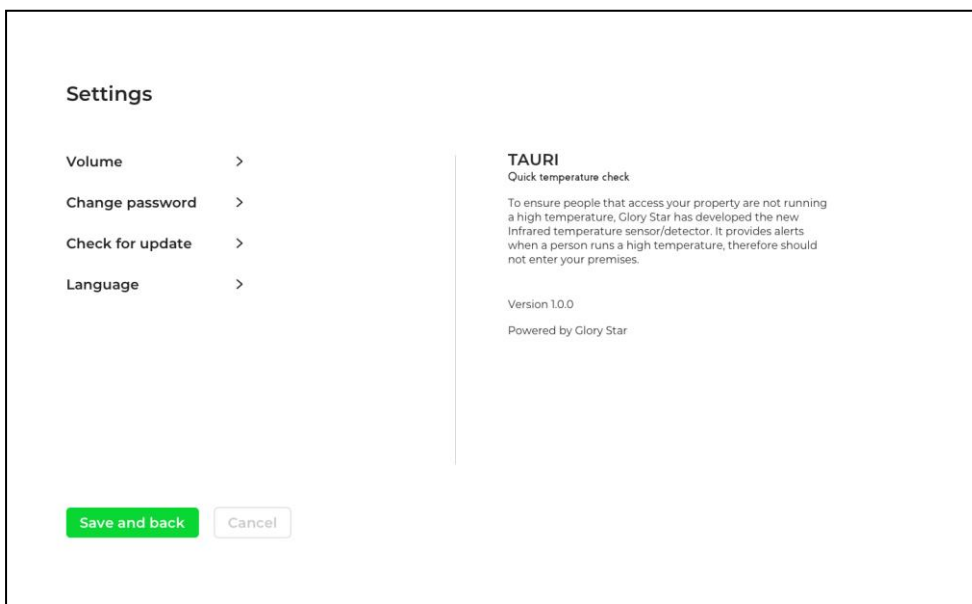


2.2 Adjust the volume, change the password and check firmware update in the settings. Press “Save and back” at the bottom left after you have finished your settings.

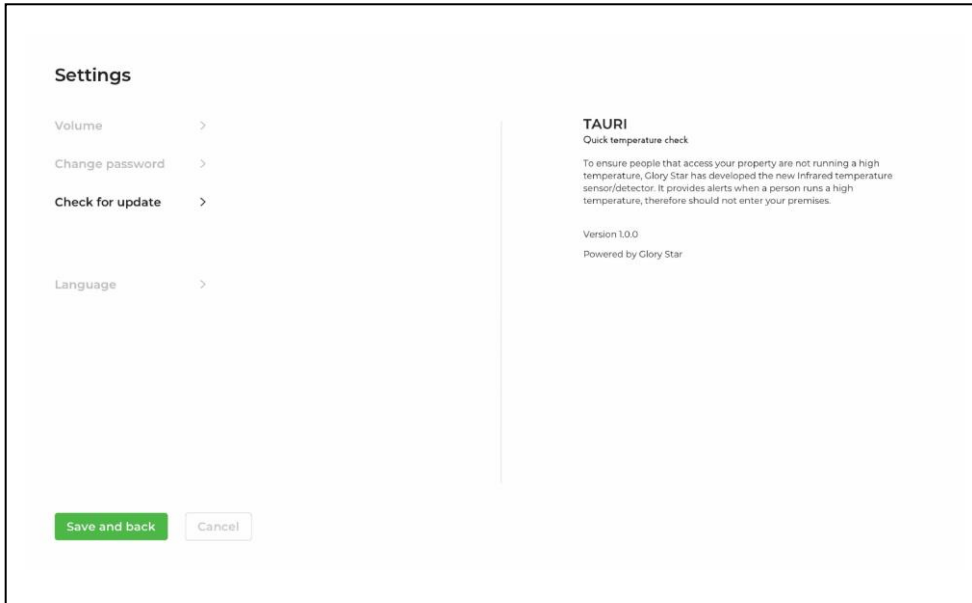


2.3 Connect to the internet in order to check and download the latest version.

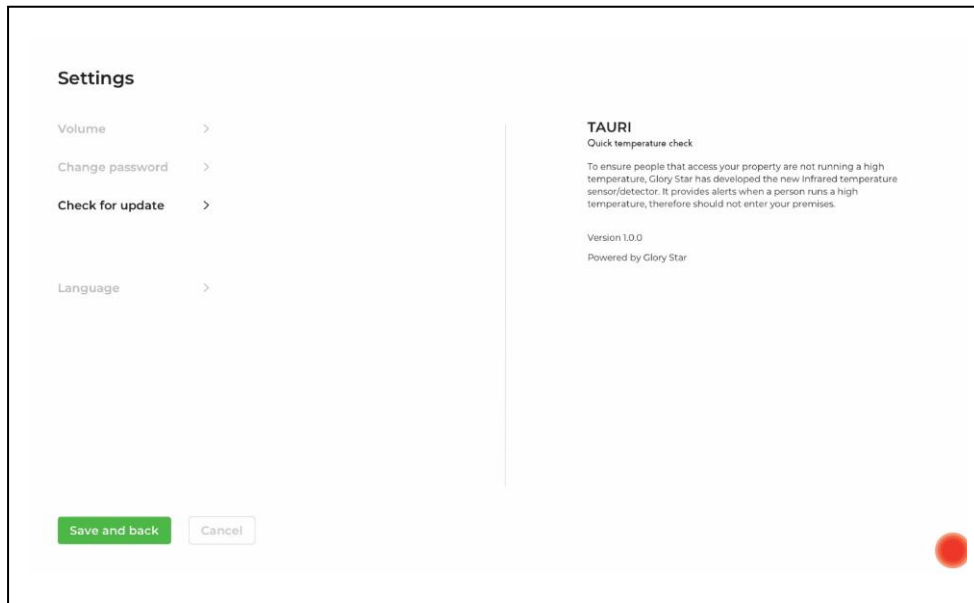


2.4 To get connected to **WIFI**, press the bottom right corner **4 times** and enter "0000" to exit TAURI.

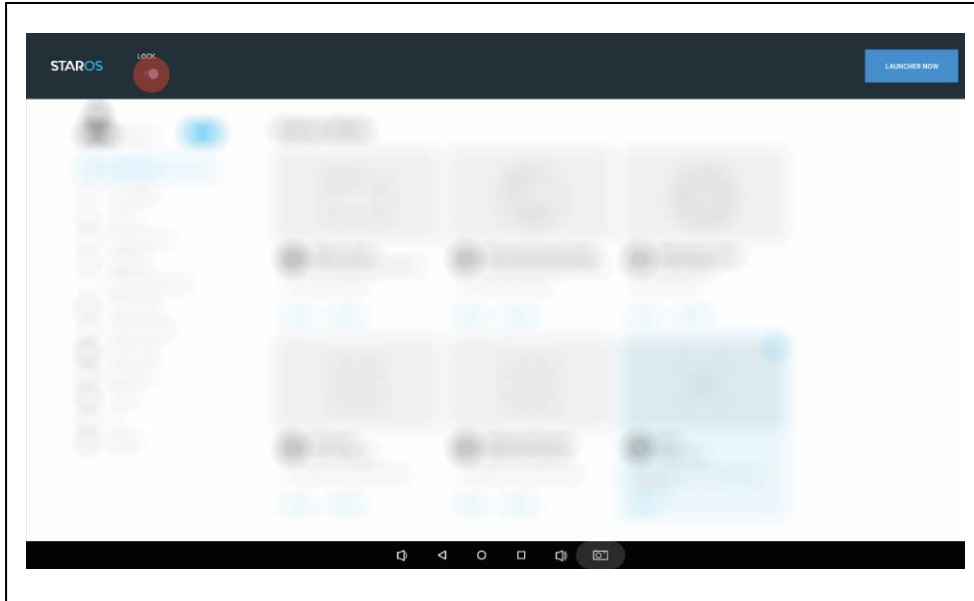


2.5 Click the icon on the top left corner to enter password.

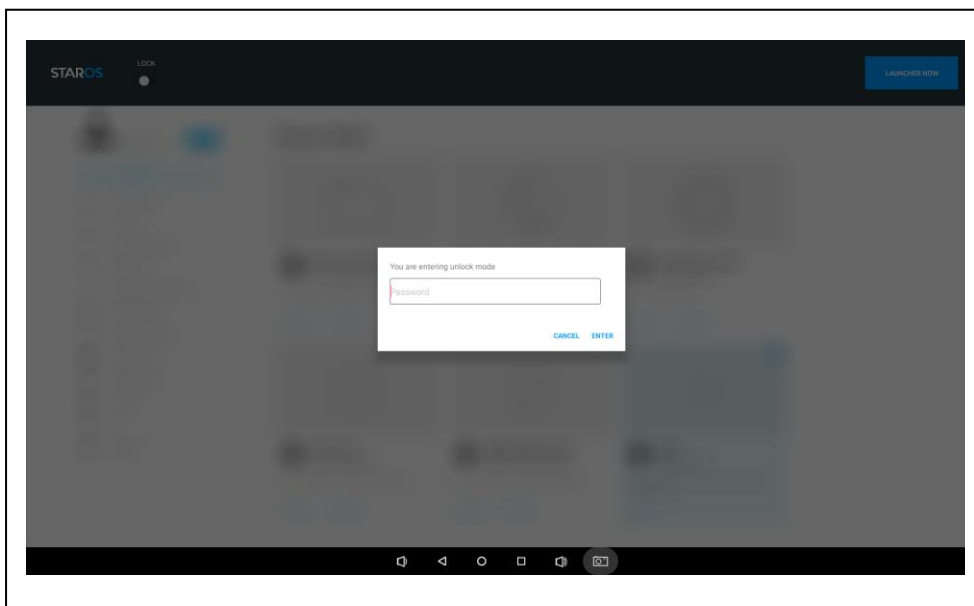


2.6 Enter password "0000" to unlock the screen.

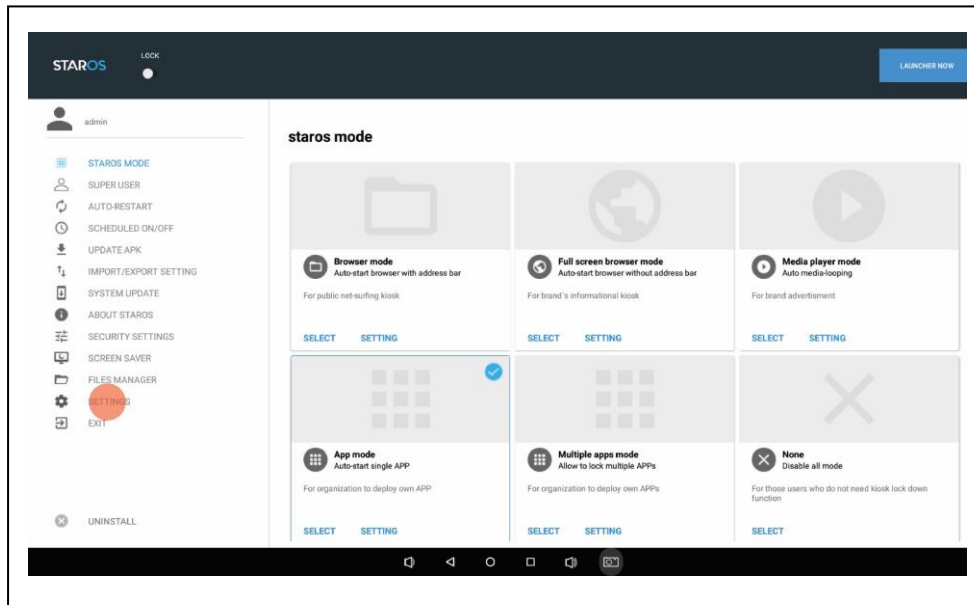


## 2.7 Choose Settings.

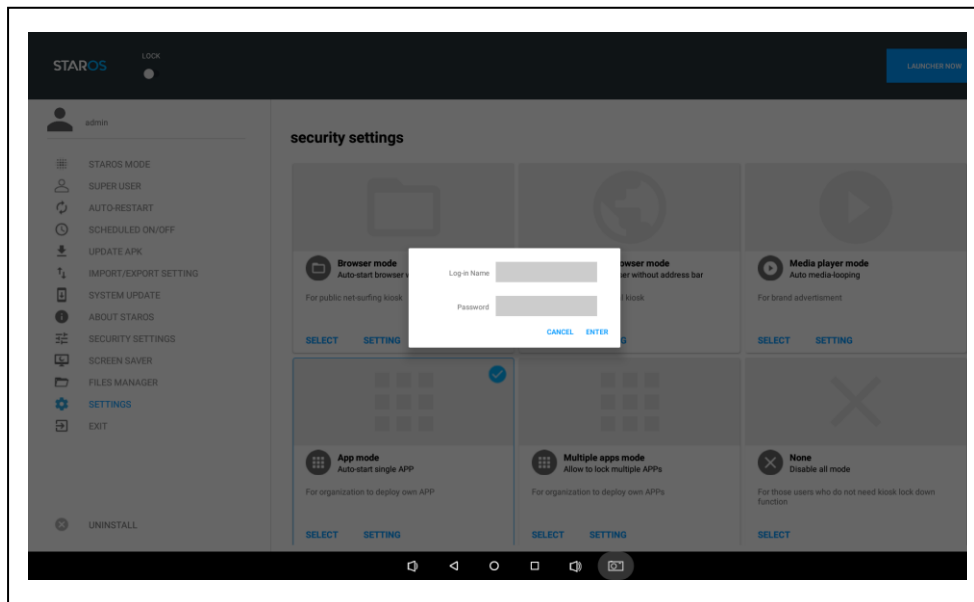

## 2.8 Enter log in name " admin " and password " 0000 ".

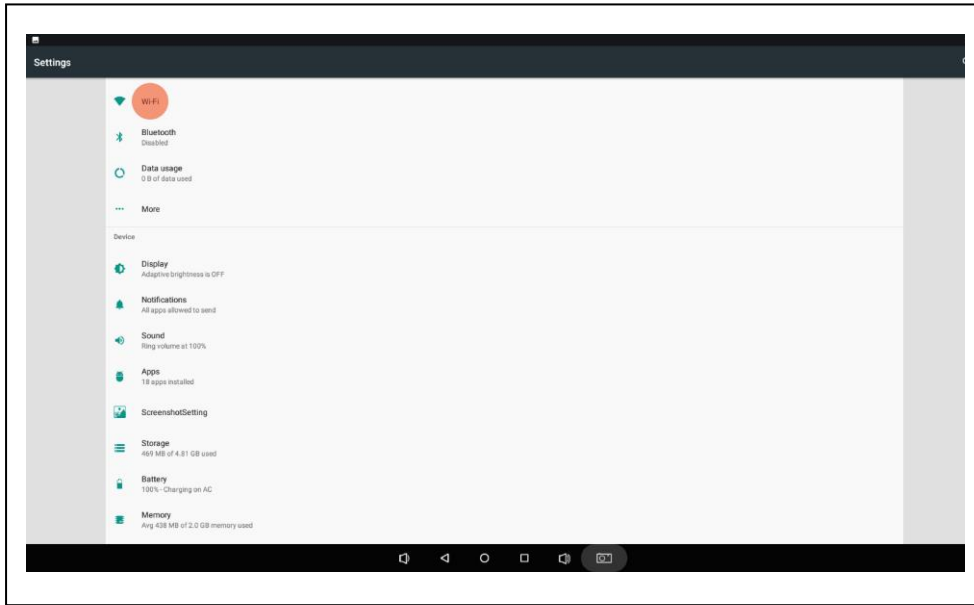


## 2.9 Select WIFI

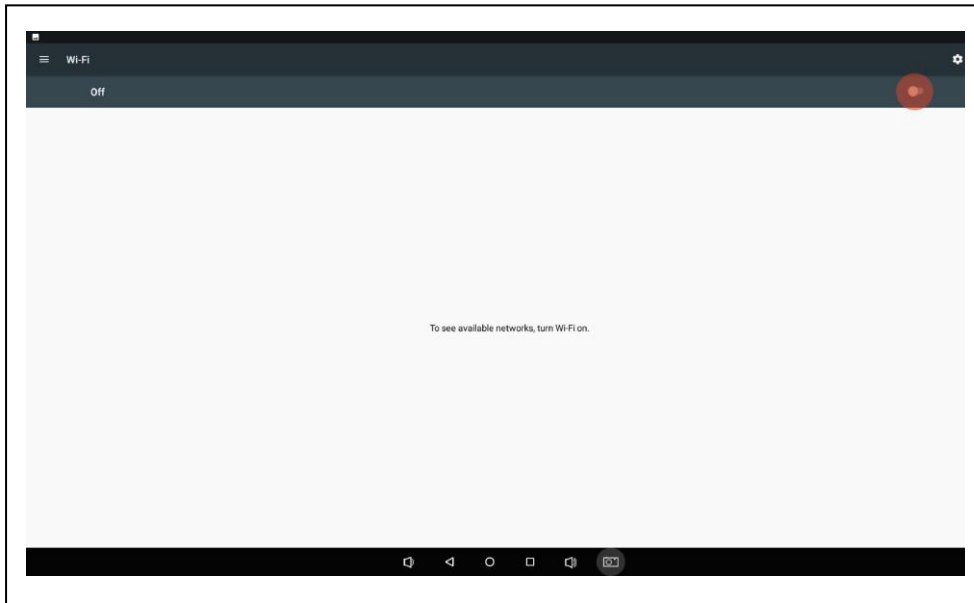


## 2.10 Select turn-on and connect WIFI




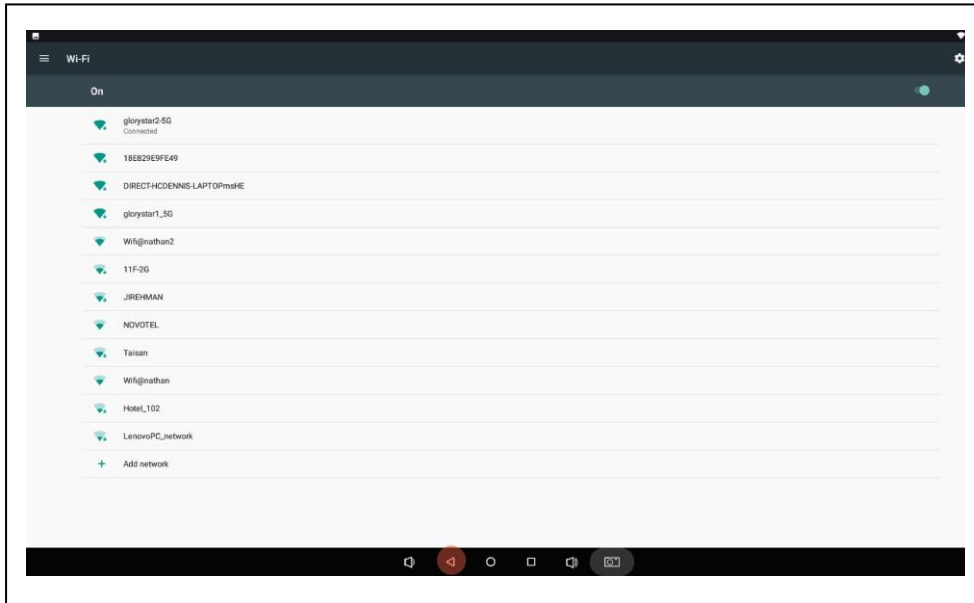

2.11 Connect WIFI and Click “back” button two times to go back to StarOS.

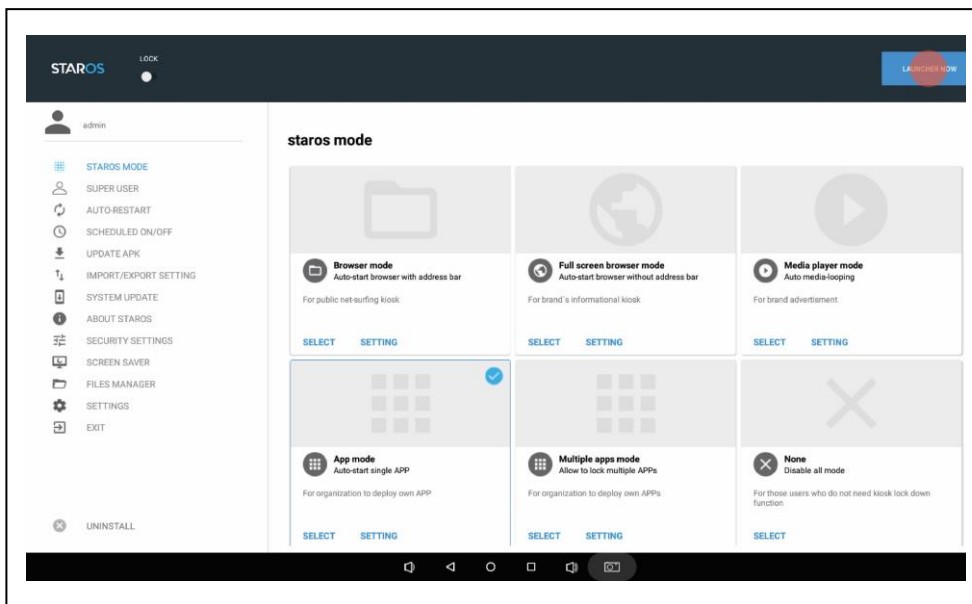


2.12 Select “Launch Now” to apply settings and return to TAURI.

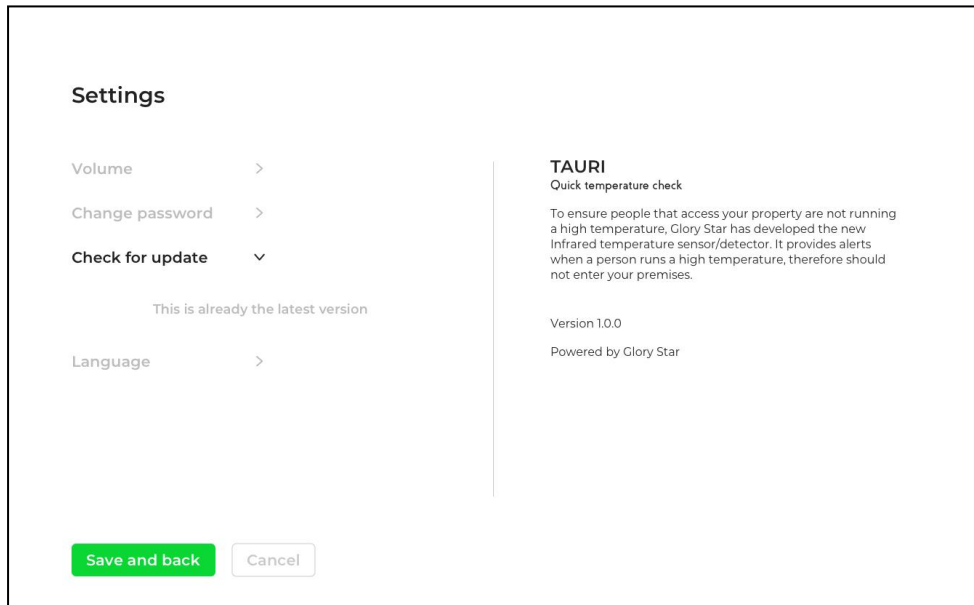


2.13 Check the version after connected to the internet successfully.

## 3. Installation


### 3.1 Wall Mount(All sizes)

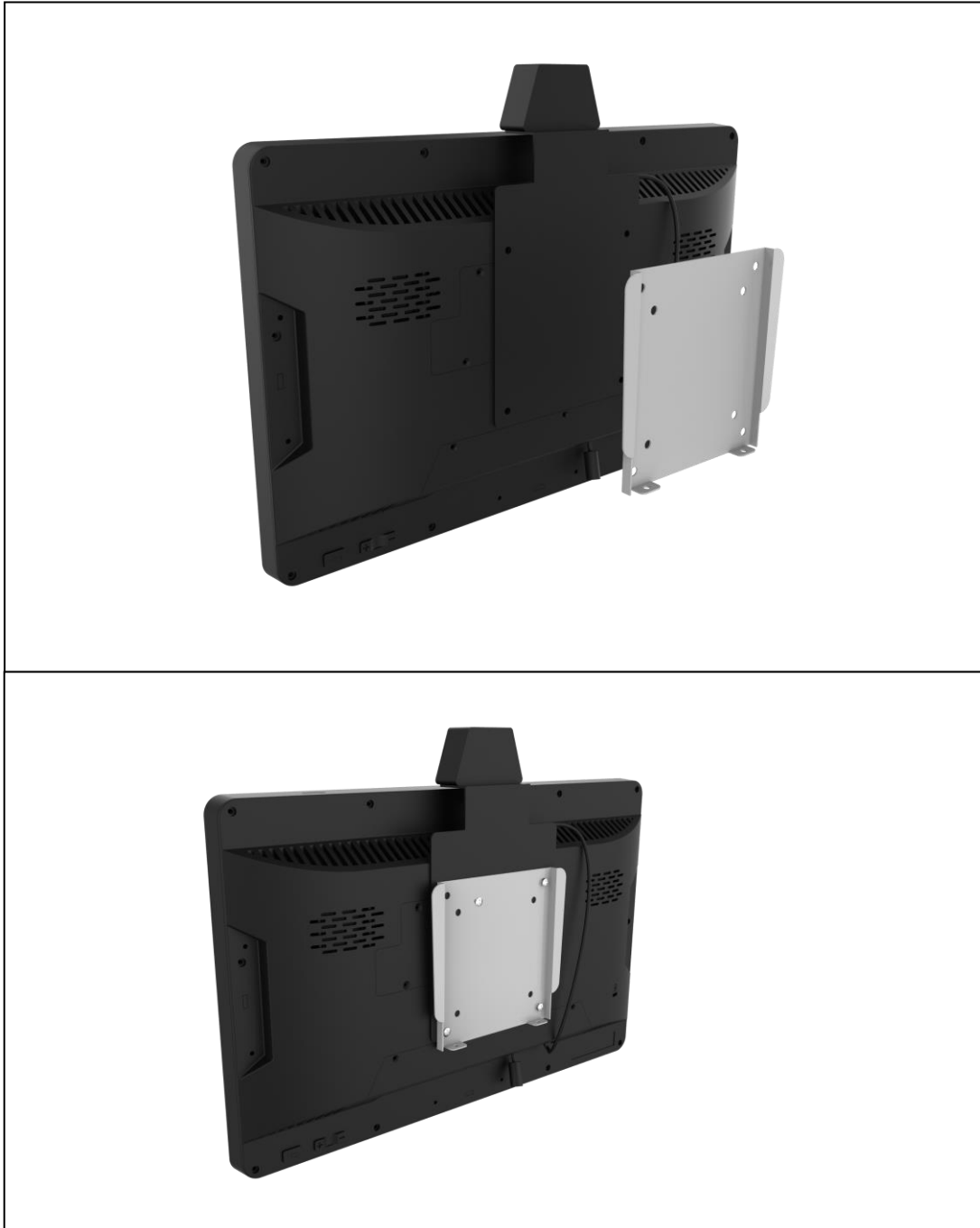

Install the wall mount to the back of the device and put screws on the 4 holes of the mount aligning to the TAURI's back mount holes.

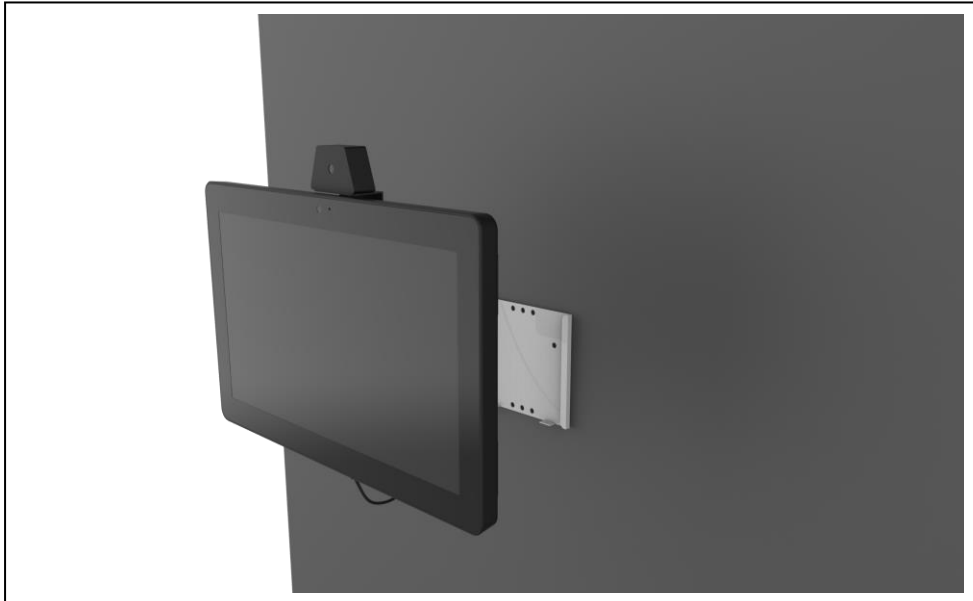

Install the second part of the wall mount to the wall.

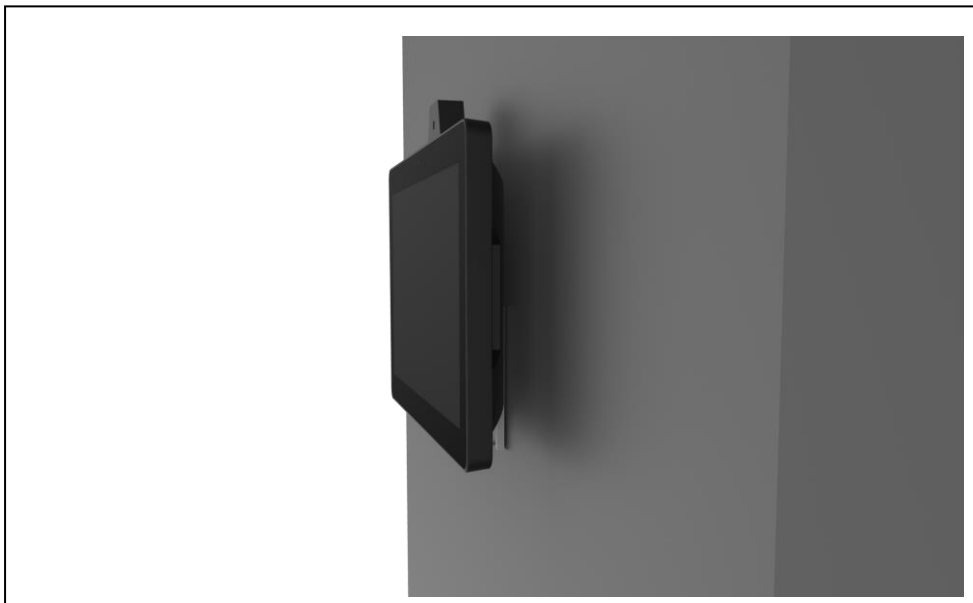

Slide TAURI with the wall mount on to the mount on wall using the hinges at the side of the wall mount.

### 3.2 Table Stand FIX 027 (10" to 21.5" TAURI)



Align the 4 screws holes at the back of TAURI to FIX-027 Table Stand screw positions, and place screws to fixate it.

### 3.3 Table Stand FIX 028 (15" to 21" TAURI)



Align the 4 screws holes at the back of TAURI to FIX-028 Table Stand screw positions, and place screws to fixate it.

### 3.4 Table Stand FIX030(All Sizes)



Align the 4 screws holes at the back of TAURI to FIX-030 Table Stand screw positions, and place screws to fixate it.

## **Glory Star Support**

Flat D, 5/F, Excelsior Building, 364 Nathan Rd., Kowloon, H.K

Hong Kong: (852) 2376 2803

USA: (626) 228 5927

E-mail: [support@glorystargroup.com](mailto:support@glorystargroup.com)